# ProShield<sup>®</sup> varnish

5% sodium fluoride varnish  
5% iger natriumfluoridlack

**Proshield Varnish** is fluoride varnish containing 50mg of sodium fluoride per 1ml. It also contains TCP and xylitol. It is applied to teeth for the purpose of delivering fluoride and treatment hypersensitivity. It has four flavors such as bubblegum, strawberry, melon and mint. **Proshield Varnish** is packed in single dose package allows easy mixing and application.

## Composition

- Rosin
- Sodium Fluoride
- Ethanol
- Xylitol
- TCP

## Indications

- For use as a desensitizing agent to help relieve dentinal hypersensitivity; To line cavity preparations under amalgam restorations
- Caries prophylaxis

## Contra Indications

1. Do not use on patients with ulcerative gingivitis or stomatitis.
2. If patients are known to be allergic to ingredients including fluoride, rosin or similar materials, **Proshield Varnish** should not be used.
3. After the application of **Proshield Varnish** patients should avoid chewing hard food, drinking liquids containing alcohol and brushing for 4 to 6 hours.

## Recommended Procedure

1. Thoroughly clean the teeth and remove excess moisture with cotton rolls and air.
2. Peel back the foil cover until the well containing **Proshield Varnish** is exposed.
3. Apply **Proshield Varnish** thinly with a brush.
4. It is recommended to remain **Proshield Varnish** on the teeth surface as long as possible at least 4 hours.

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